



SECULIFE | PS300 UNIVERSAL PATIENT SIMULATOR

3-349-769-03 1/1.14

SECULIFE PS300 for testing patient monitors

- Drop down choice screens, list all options for parameters
- MINI-DIN connectors for BP cables
- Flash-programmable for upgrades
- Full remote operation via RS-232
- No codes to remember or enter
- · Programmable sequences
- Simple to operate



The SECULIFE PS300 is an easy-to-use patient simulator in which all parameters can be selected through a graphic display. Autosequences for BDM and printing can be selected. 10 Patient connections are available. An extension to SpO2 function is possible. Two channels for blood pressure and 49 arrhythmias are available.

ECG Waveforms

The microprocessor has stored in its memory all of the digitalized waveforms. It sends the individual lead waveforms to D/A converters, which generate accurate analog representations. The waveforms are then sent through resistor networks, developing the appropriate signals on the output terminals.

Cardiac Output

The SECU**LIFE** PS300 has the ability to simulate Thermodilution Cardiac Output measurements. Thermodilution allows the calculation of heart volume output by measuring the temperature change of the blood after a specific volume of solution, which is room temperature or cooler (typically 0 °C or 24 °C), is injected into the heart. The blood temperature is measured by a thermistor on the catheter.



Purchase parts

- Operating Instruction
- 1 Transport case
- 2 Batteries 9 V DC
- 1 Power supply (model U.S.)
- 1 Power supply (model EU)

SECULIFE | PS300 UNIVERSAL PATIENT SIMULATOR

Technical Data

Device:

Construction	227.8 x 153.4 x 43.7 mm/ABS Plastik
Weight	< 2 Lbs (0.91 kg)
Face Plate	Lexan/Back printed
Operating Range	15 to 40 °C
Storage Range	-20 to 65 °C

General		
Display	1LCD graphic, 128 x 64 pixel	
Backlight	Yes	
Lead Test Terminals	Yes	
RS232	Yes	
Power	2 x 9 V Batteries	
Battery Eliminator	Without SECULIFE OX 9 V DC, 200 mA With SECULIFE OX 10 V DC, 500 mA	

Functions:

ECG-NSR	Yes	
Leads	10	
Rates	17	
Amplitudes	19	
QRS Interval	Adult + Pediatric	
ST Segment Elevation	19	
ECG-Performance	Yes	
Sine Waves	8	
Square Waves	2	
Triangle Waves	2	
Pulse Waves	3	
Respiration	Yes	
Rates	8	
Baseline	4	
Delta Impedance	6	
NIPB	No	
Rates	-	
Pressure Output	-	
Special Modes	Yes	
SpO2	With Seculife OX	
Fetal/Maternal	Option	
Cardiac Output	Option	
Trainings Mode	Yes	

Pacemaker	No		
Arrhythmias	49		
Blood Pressure	2		
	YSI 400/700		
Temperature	0, 24, 30, 37, 40 °C		

Ordering information

Description	Туре	Order number
Universal Patient Simulator	SECU LIFE PS300	M695N

For more information on accessories, see

• www.gossenmetrawatt.com

Edited in Germany • Subject to change without notice • A PDF version is available on the Internet

Phone +49 911 8602-111 Fax +49 911 8602-777 E-Mail info@gossenmetrawatt.com www.gossenmetrawatt.com